

Hugs Infant Protection

Swaddled in security

You do so much to protect the patients in your care, let us help.

Screen captures are for illustrative purposes only and may slightly vary at your location.

What is Hugs?

Hugs is an infant protection system for hospitals, and has a “safe area” where babies are monitored at all times, it consists of:

<p>Hugs Tags</p>		<p>Hugs tags are applied the baby.</p>
<p>Exciters</p>		<p>Exciters are installed by every exit and will alert when babies are detected nearby the entrance.</p>
<p>Receivers</p>		<p>Receivers are installed throughout the unit and detect the status of the tag.</p>
<p>Client PC</p>		<p>The Client PC runs the Hugs System. It contains a map of your facility and shows the current status of every active Tag. It also displays warning messages when alarms occur.</p>

Hugs Security Features

Hugs generates alarms when babies are removed from the safe area, when a tag has been tampered with or is not in contact with the infant.

Alarm Type	An Alarm occurs when...
Exit Notification	An infant Hugs tag is close to or passes through an open door.
Cut Band or Tamper Technology	The Hugs tag is damaged or cut without authorization.
Supervision	The system does not “hear” a Hugs tag “check-in” for configured amount of time.

Hugs Tag



- Is a reusable tag that secures around a baby's ankle using a disposable strap.
- It can only be admitted on a dedicated Hugs unit.
- The tag sends out a regular Heartbeat™ when it's on the unit.



IT WILL SEND OUT AN ALARM WHEN:

- it's removed from the unit
- when it's tampered with
- when the tag is loose
- and when the battery power is low.

Admitting tags is as easy as 1,2 3!



You will need:

1 Hugs Tag


1 Tamper Strap

A pair of safety scissors

And a baby!

Important Note: Attach tags only within your hospitals safe area



Step 1: Insert the Strap

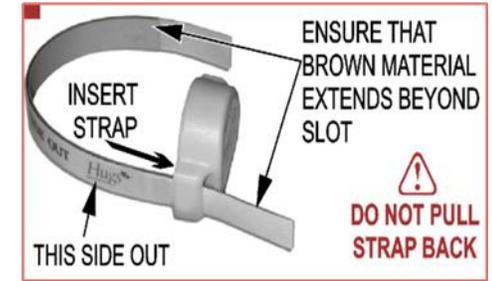

STEP	ACTION
1	Align strap with tag slot, making sure “This side out” in on the outside.
2	Insert strap into tag slot starting from bottom of Tag.
3	Pull through until the brown material extends beyond the slot. You will hear a two-tone beep verifying the tag is active

Remember: You want it loose enough to wiggle but not rotate on the ankle.

Step 2: Place the tag on the baby

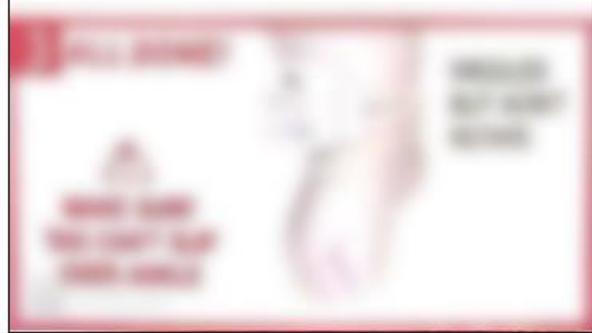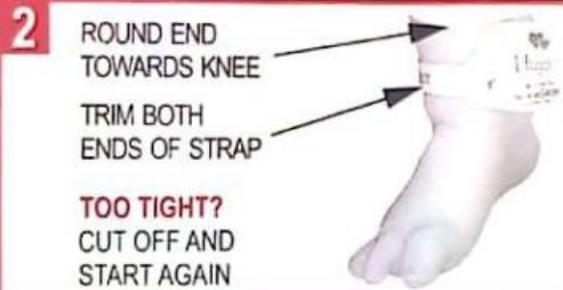

STEP	ACTION
1	Wrap the tag around baby's ankle and insert loose end of the strap into the second tag slot.
2	Pull the strap through until the Tag is snug but still wiggles.
3	Trim both ends of the strap with scissors.

Step 3: Final Check



STEP	ACTION
1	<p>Perform a final check for the following:</p> <ul style="list-style-type: none">❖ Tag is not too tight and wiggles but won't rotate.❖ Tag cannot be slipped over ankle or foot.❖ Tag has been admitted properly in the system.
2	<p>If the tag admitted properly, you will have heard a two-tone beep verifying the tag is active.</p> <p>If the tag has not been properly admitted proceed to troubleshooting steps.</p>

If the tag has not properly admitted...

Check the following:

- Is the brown material on the strap touching the contacts on both sides of the Tag?
- Has the Tag been attached within the area covered by Hugs?
- Has the battery expired? A warning appears in the software.
- Has the strap been damaged? Is the brown material torn?

Tightening the Hugs Tag

Tighten a Tag when the baby has lost weight to make sure it remains snugly attached.

Work strap through from the bottom using a side-to-side rocking motion until there is enough strap to pull on.

Snug up Tag until it is secure but still wiggles freely.

Trim strap.

If it appears too tightly applied



DO NOT try to “loosen” the tag by pulling the strap back.

This will cause a small tear in the brown material, and you will get nuisance tag tamper alarm. If the tag is too tight, suspend tamper and place a new band on.

Client PC Details – Hugs software



Login

User Name:

Password:

STANLEY Healthcare Hugs

Alarms-Events Census (5) Reports Admin Hello tpc Log out

Name	ID	Room	Gender	Admitted	Suspend	Transport	Memo	Zone
Test 1	681	519	Boy	11/19/2018 11:28:50 AM				5th Floor
Test 2	632	Loria Nursery	Girl	11/19/2018 11:56:16 AM				5th Floor
Test 3	672	Loria Nursery	Ambiguous	11/21/2018 1:53:17 PM				5th Floor
Test 4	953	NICU 1	Ambiguous	11/26/2018 9:27:06 AM				5th Floor
Test 5	677	T5 Nursery	Boy	11/26/2018 10:57:42 AM				5th Floor

STANLEY Healthcare Hugs

Alarms-Events (3) Census (5) Reports Admin Hello tpc Log out

Name	ID	Room	Gender	Admitted	Suspend	Transport	Memo	Zone
Test 4	953	NICU 1	Ambiguous	11/26/2018 9:27:06 AM				5th Floor
Test 5	677	T5 Nursery	Boy	11/26/2018 10:57:42 AM				5th Floor
Test 1	681	519	Boy	11/19/2018 11:28:50 AM				5th Floor
Test 2	632	Loria Nursery	Girl	11/19/2018 11:56:16 AM				5th Floor
Test 3	672	Loria Nursery	Ambiguous	11/21/2018 1:53:17 PM				5th Floor

STANLEY Healthcare Hugs

Alarms-Events Census (5) Reports Admin Hello tpc Log out

Name	ID	Room	Gender	Admitted	Suspend	Transport	Memo	Zone
Test 1	681	519	Boy	11/19/2018 11:28:50 AM				5th Floor
Test 2	632	Loria Nursery	Girl	11/19/2018 11:56:16 AM				5th Floor
Test 3	672	Loria Nursery	Ambiguous	11/21/2018 1:53:17 PM				5th Floor
Test 4	953	NICU 1	Ambiguous	11/26/2018 9:27:06 AM				5th Floor
Test 5	677	T5 Nursery	Boy	11/26/2018 10:57:42 AM				5th Floor

Connected: tpc-hps-021 11:02 AM

Screen Tabs

To access the different Hugs® screens (Census, Alarms-Events etc.), select the appropriate screen tab located at the top of the Hugs window.

Select a screen tab to access the different screens



Admitting the Hugs Tag *(continued...)*

STEP	ACTION
1	Once the strap has been pulled through both slots, a dialog box will appear on the Hugs computer screen and the tag will auto admit and supervision will take place.
2	Go through each section line by line. The tag number will display and should be verified for correctness. Change the descriptions as soon as possible.
3	DO NOT leave any line blank. This information is necessary to track the tag. When done click OK to save. The tag will be monitored by the system while in the protected area. Alarms will occur if the tag is tampered with, or an unauthorized exit is attempted.

The screenshot shows a software dialog box titled "Admit". It contains the following fields and values:

- Tag ID:** 987
- Name:** (empty text box)
- Zone:** (empty dropdown menu)
- Gender:** Labor and Delivery (selected), Maternity (available in dropdown)
- Room #:** (empty dropdown menu)

At the bottom of the dialog box are two buttons: "OK" and "Cancel".

Note:

After a discharge, auto-admit is disabled for about 45 minutes.

Alarm Message

STANLEY Healthcare

Alarms-Events (1) Census (3)

5th Floor St. Raphael Hospital

5th Flr Stair LSST1

Maternity Main Desk

5th Flr Main Bldg Stair MSST2

Hugs Server VM

5th Flr Staff Elevators

5th Flr OR Corridor Exit

5th Flr Stair MSST1

L & D Main Desk

5th Flr Main Elevators

NICU Desk

Alarms-Events (1)

Description	Type	Da
Hugs Tag 956	Check tag tightness	10/17

Details

Strongest LAR 1: 5th Flr NICU (17 secs ago)
 Strongest LAR 2: 5th Flr Main Elevators (5 secs ago)
Gender:
Room Number:

Procedure

P.T.S SKINS NOT MAKING CONSISTENT CONTACT WITH THE P.T.S SKIN

1. Immediately verify patient's location/safety.
2. Clear alarm and tighten Tag, as appropriate.
3. If pt. cannot be located, initiate Code Amber.

Alarm Notes

Select note from list or enter a reason

Mute Clear

Connected (huasrc1vp) 7:23 AM

Alarm Message Window



Details Window



Procedures Window



Type Alarm Notes here >>

Alarms-Events (1)

Description	Type	Da
Hugs Tag 956	Check tag tightness	10/17

Details

Strongest LAR 1: 5th Flr NICU (17 secs ago)
 Strongest LAR 2: 5th Flr Main Elevators (5 secs ago)
Gender:
Room Number:

Procedure

P.T.S SKINS NOT MAKING CONSISTENT CONTACT WITH THE P.T.S SKIN

1. Immediately verify patient's location/safety.
2. Clear alarm and tighten Tag, as appropriate.
3. If pt. cannot be located, initiate Code Amber.

Alarm Notes

Select note from list or enter a reason

Mute Clear

7:23 AM

Alarms Troubleshooting

Handout provided and procedures will be displayed in your Hugs Alarm Window

Alarm Type:

- When do they occur?
- What happens?
- How do I respond?

Alarm Type	When do they occur?	What happens?	How do I Respond? Follow your hospital's procedure and:
Loose Tag	When the tag is too loose on the infant.	<ul style="list-style-type: none"> An alarm message appears in the Hugs software. 	<ul style="list-style-type: none"> Inspect the tag and make sure it is properly attached to the infant. Inform your System Administrator or Security. Check tags every hour and tighten as needed.
Exit	A baby wearing a Tag is removed from safe area without being signed out or discharged or is brought near a doorway when door is open without being signed out or discharged.	<ul style="list-style-type: none"> An alarm message appears in the software. An alarm may sound. 	<ul style="list-style-type: none"> Confirm whether the baby has been taken from safe area. Help locate the baby.
Tamper	The tag has been tampered with.	<ul style="list-style-type: none"> An alarm message appears in the software. An alarm may sound. 	<ul style="list-style-type: none"> Locate the baby and check Tag for tampering. Replace strap or Tag if necessary.
Improperly Applied Strap	The tag is not applied properly.	<ul style="list-style-type: none"> An alarm message appears in the software. An alarm may sound. 	<ul style="list-style-type: none"> Inspect strap and engage Tamper mechanism properly. Replace strap or Tag.
Strap Detached	When a Tamper or Improperly Applied Strap Alarm is cleared, but the tag is still in Tamper mode.	<ul style="list-style-type: none"> An alarm message appears in the software. An alarm may sound. 	<ul style="list-style-type: none"> Inspect strap and engage Tamper mechanism properly. Replace strap or Tag.
Auto Discharge	When a Tamper or Improperly Applied Strap Alarm is cleared, and the tag has gone into Strap Detached Alarm. If the strap has not engaged the Tag will auto discharge.	<ul style="list-style-type: none"> An alarm message appears in the software. The tag will appear in the Census – Greyed Out! 	<ul style="list-style-type: none"> Replace the infant's tag as a new admission.
Supervision	Tag's Heartbeat has not been seen by software for a set time. System hardware is not being seen by network devices.	<ul style="list-style-type: none"> An alarm message appears in the software. An alarm will sound. 	<ul style="list-style-type: none"> Report the event to security. Provide notes for follow-up. Help locate the baby. Remove the Tag and set aside for evaluation.
Transport Time Expired	When a Tag has been gone from the system for longer than the Sign Out time.	<ul style="list-style-type: none"> An alarm message appears in the software. An alarm may sound. 	<ul style="list-style-type: none"> If the baby has been returned to the safe area, sign him/her in. If the baby has not been returned to the safe area, verify his/her location and then extend the Sign Out period.
Duplicate Tag ID	When the system detects a Tag with the same ID number as a Tag already admitted.	<ul style="list-style-type: none"> An alarm message appears in the software. An alarm may sound. 	<ul style="list-style-type: none"> Locate both babies. Discharge the Tag and remove the Tags from both babies. Attach a new Tag to both babies. Set aside the duplicate Tag and inform your System Administrator.
Low Battery	When the battery level becomes low.	<ul style="list-style-type: none"> An alarm message appears in the software. 	<ul style="list-style-type: none"> When the Tag is removed, separate it from Tags to be reused. Inform Security or your System Administrator.

*Very rarely you may see an alarm for a system device, such as a Patient Security Receiver or Exciter. Contact your System Administrator or Security for guidance.

The Census Screen

The screenshot shows the Hugs Healthcare software interface. At the top, there are tabs for 'Alarms-Events', 'Census (5)', 'Reports', and 'Admin'. The 'Census (5)' tab is active. Below the tabs is a toolbar with various icons. The main area displays a table with the following columns: Name, ID, Room, Gender, Admitted, Suspend, Transport, Memo, and Zone. The table contains five rows of test data. A right-click context menu is open over the 'Test 5' row, showing options: History, Discharge, Transport, Suspend Tamper, Edit Memo, Locate, and Disable BabySense.

Name	ID	Room	Gender	Admitted	Suspend	Transport	Memo	Zone
Test 1	681	519	Boy	11/19/2018 11:28:50 AM				5th Floor
Test 2	632	Loria Nursery	Girl	11/19/2018 11:56:16 AM				5th Floor
Test 3	672	Loria Nursery	Ambigious	11/21/2018 1:53:17 PM				5th Floor
Test 4	953	NICU 1	Ambigious	11/26/2018 9:27:06 AM				5th Floor
Test 5	677	T5 Nur...	Boy	11/26/2018 10:57:42 AM				5th Floor

Right- Click for tag options

Census List Row Colors


The different row colors that are used in the Hugs Census list are shown here:

Name	Color	Meaning
Major, girl	White	Normal activity
Smith, boy	Red	Alarm state
Taylor, boy	Yellow	Transported
Peters, boy	Orange	Tamper suspended
Charles, girl	Blue	BabySense disabled
Lolly, girl	Gray	Auto-discharged
Grondin, girl	Brown	Tag low battery

Hugs Functions

Authorized users can perform a function for a Hugs tag using one of the following methods:

1) Toolbar Icons

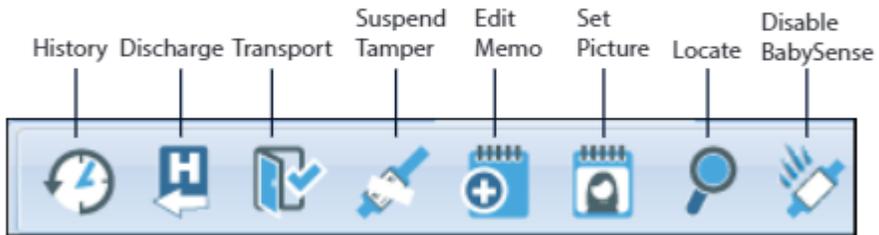

- On the Census List, click on the tag/infant row to highlight it.
- Click the toolbar icon representing the requested function.

2) Popup Menu

History
Discharge
Transport
Suspend Tamper
Edit Memo
Set Picture
Locate
Disable BabySense

- On the Census List, click the red heart icon beside the appropriate tag/infant name.
- Select a Hugs function from the popup menu.

Note: The toolbar icons and popup menu commands may vary depending on your access permissions and how your system was set up.

History

STANLEY Healthcare Hugs[®]

Alarms-Events (0) Censu (5) Reports Admin

Name	ID	Room	Gender	Admitted	Suspend	Transport	Memo	Zone
Test 4	953	NOCU 1	Ambiguous	11/26/2018 9:27:06 AM				5th Floor
Test 5	677	TS Nursery	Boy	11/26/2018 10:57:42 AM				5th Floor
Test 1	681	S19	Boy	11/19/2018 11:28:50 AM				5th Floor
Test 2	632	Loria Nursery	Girl	11/19/2018 11:56:16 AM				5th Floor
Test 3	672	Loria Nursery	Ambiguous	11/21/2018 1:53:17 PM				5th Floor

History

- Discharge
- Transport
- Suspend Tamper
- Edit Memo
- Locate
- Disable BabySense

History (Test 3)


Event	Type	Details	User	PC	Date/Time
Log	Tag Transport Return	Authorized Return	tpc	BSCGSHCTR...	11/26/2018 11:41:01 AM
Log	Tag Transport	Transport for 00 hour(s) 15 min(s) 00 sec(s) (Zones: 5th Floor)	tpc	BSCGSHCTR...	11/26/2018 11:39:00 AM
Log	Tag Transport Return	Authorized Return	TMcquaid	BSCGSHCTR...	11/26/2018 10:45:34 AM
Log	Tag Transport	Transport for 00 hour(s) 15 min(s) 00 sec(s) (All Zones)	TMcquaid	BSCGSHCTR...	11/26/2018 10:45:06 AM
Log	Setting Changed	Room value changed from (Old Nursery) to (Loria Nursery).	tpc	BSCGSHHUG...	11/26/2018 9:26:17 AM
Log	Setting Changed	Description changed to Test 3	Jgunder	BSCGSHHUG...	11/26/2018 9:20:06 AM
Log	Setting Changed	Room value changed from () to (Old Nursery).	tpc	BSCGSHCTR...	11/26/2018 9:07:15 AM
Configure	Setting Changed	BabySense enabled for the tag.	tpc	BSCGSHCTR...	11/26/2018 9:06:25 AM
User Clear	Tag Exit Alarm		Jgunder	BSCGSHHUG...	11/21/2018 2:00:18 PM
Alarm	Tag Exit Alarm	Alarm Location: 5th Floor Lobby Elevs 3 & 4; Room: Gender: Ambiguous;			11/21/2018 1:58:53 PM
Log	Tag at Exit	Hugs Tag 672 at 5th Floor Lobby Elevs 3 & 4			11/21/2018 1:58:53 PM
Log	Memo Entered		Jgunder	BSCGSHHUG...	11/21/2018 1:57:06 PM
User Clear	Check tag tightness		Jgunder	BSCGSHHUG...	11/21/2018 1:54:30 PM
Event	Check tag tightness	Room: Gender: Ambiguous;			11/21/2018 1:54:06 PM
Configure	Setting Changed	BabySense disabled for the tag.	Jgunder	BSCGSHHUG...	11/21/2018 1:54:05 PM
Log	Tag Admitted	Dialog acknowledged	Jgunder	BSCGSHHUG...	11/21/2018 1:53:52 PM
Log	Setting Changed	Gender value changed from () to (Ambiguous).	Jgunder	BSCGSHHUG...	11/21/2018 1:53:52 PM
Log	Tag Off-Skin Messages				11/21/2018 1:53:27 PM
Log	Tag On-Skin Messages				11/21/2018 1:53:19 PM
Log	Tag Admitted				11/21/2018 1:53:17 PM
Log	Zone Change	Zone changed from " " to '5th Floor'			11/21/2018 1:53:17 PM

Close


Discharge

STEP	ACTION
1	Right click on the baby's name in the census screen.
2	Select discharge. A warning box will appear on the screen. Select "OK".
3	Cut the strap using safety scissors. <i>Tip: Remove half the strap. Leave one half indicating a dirty tag. You do not want the tag to make contact again.</i>
4	Set aside the tag for cleaning.


Note: The Hugs Tag must be cleared from the software before you physically remove it. This is called "discharging the infant."

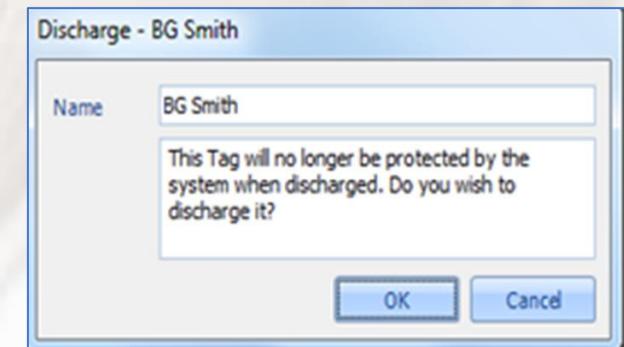

Discharging a Tag

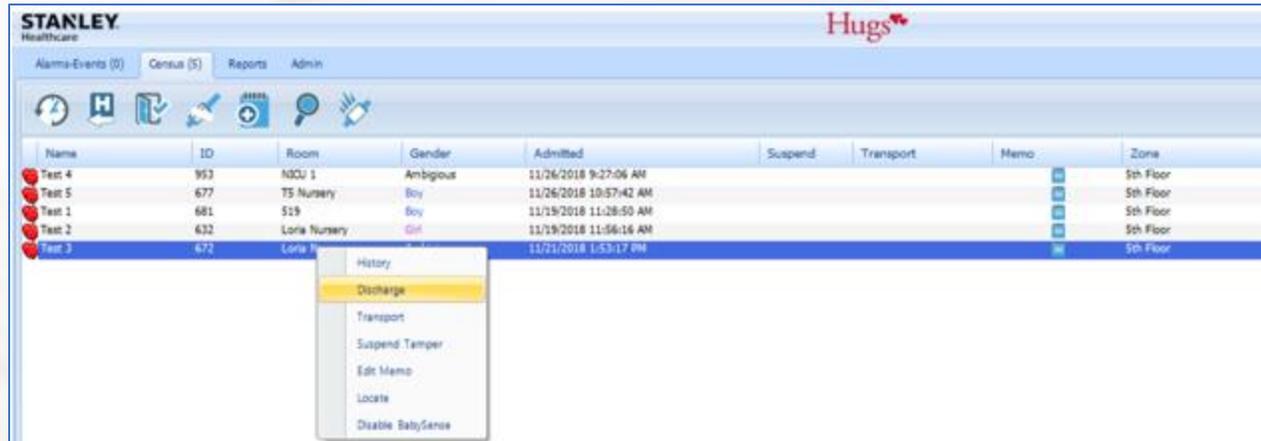

Discharging is the expression used to indicate the removal of a Hugs tag from the Hugs system. This should typically only occur when and infant is being taken home from the hospital.

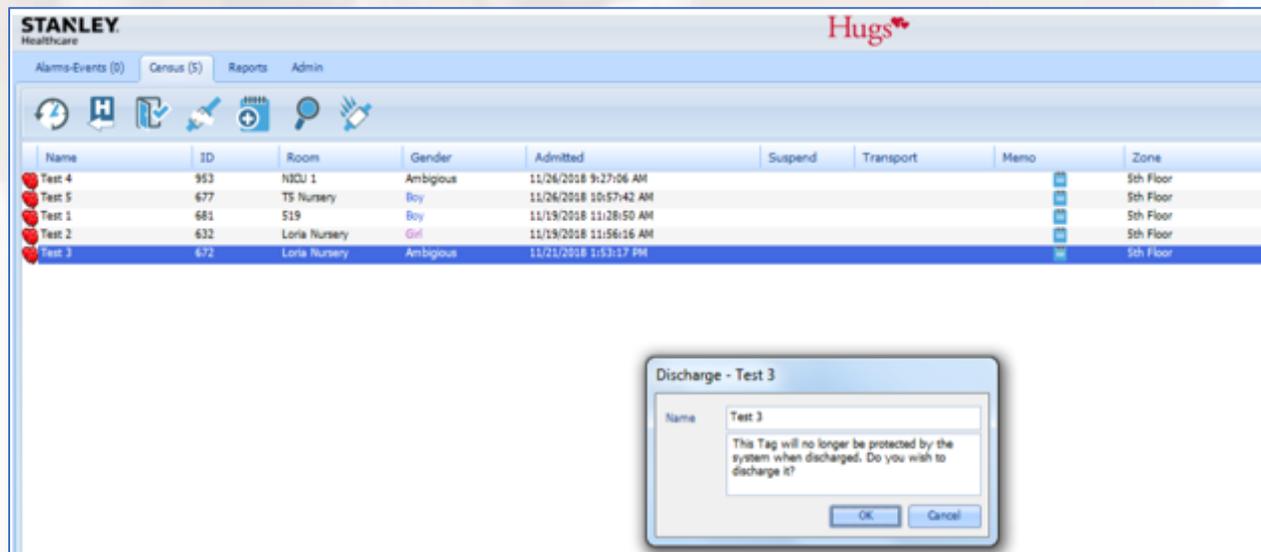


Note: *Once a tag is discharged, it continues to transmit for about 10 minutes (the Server PC ignores these messages), and then goes into "sleep" mode ready to be used again. The tag can be reused within 10 minutes but will not Auto Admit. You must manually admit it.*

Caution : Gray – Auto Discharged

Census List Row Colors
The different row colors that are used in the Hugs Census list are shown here:

Name	Color	Meaning
Major, girl	White	Normal activity
Smith, boy	Red	Alarm state
Taylor, boy	Yellow	Transported
Peters, boy	Orange	Tamper suspended
Charles, girl	Blue	BabySense disabled
Lolly, girl	Gray	Auto-discharged
Grondin, girl	Brown	Tag low battery

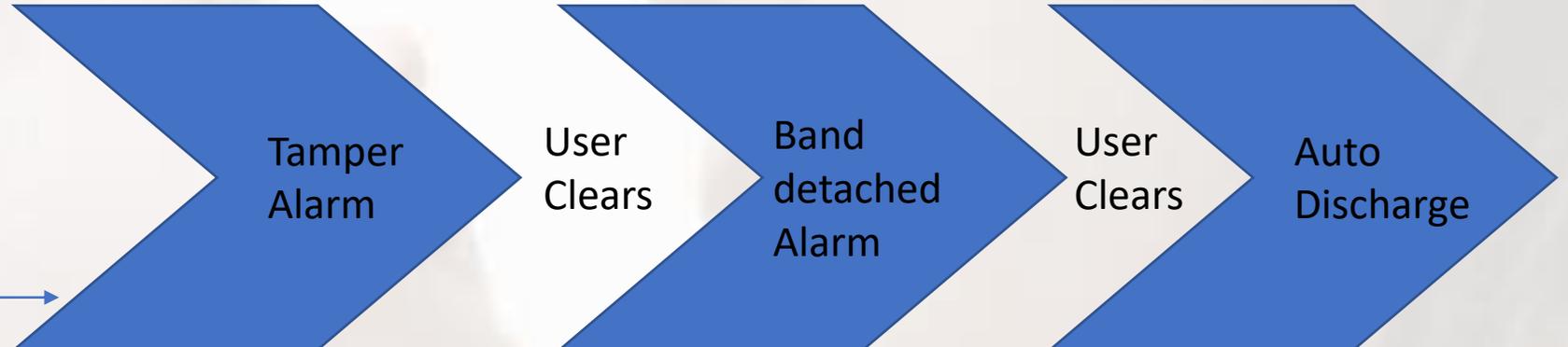

STANLEY Healthcare Hugs[®]

Alarms-Events (0) Census (3) Reports Admin

Name	ID	Room	Gender	Admitted	Suspend	Transport	Memo	Zone
Test 2	632	S12	girl	11/19/2018 11:56:16 AM				5th Floor
Test 3	672	Loria Nursery	Ambiguous	11/21/2018 1:53:17 PM				5th Floor
baby love	679	LDR 01	boy	11/26/2018 2:07:54 PM				5th Floor

STANLEY Healthcare Hugs[®]

Alarms-Events (0) Census (4) Reports Admin

Database: Active Database

Hugs Tags Memos Records: 1

Rec#	DateTime	Event	Type	Detail	Tag	ID	Location	User	PC	Memo
164563	11/26/2018 10:58:47 AM	Memo Entered	Log		Hugs Tag 677	677		tpc	BSCGSHCTR01	On tack inside nursery foyer

Auto Discharge - test 6

Name: test 6

This Tag will no longer be protected by the system when discharged. Do you wish to discharge it?

OK

An Auto Discharge Alarm Event will display on the computer and the baby's name will appear in Gray.

The next morning any names in gray will be removed from the census automatically.

Transport

The transport command allows an infant to be transported out of the protected area temporarily for tests or other purposes without generating an Exit alarm.

Initiating Transport

Name	ID	Room	Gender	Admitted	Suspend	Transport	Memo	Zone
Test 1	681	519	Boy	11/19/2018 11:28:50 AM				5th Floor
Test 2	632	Loria Nursery	Girl	11/19/2018 11:56:16 AM				5th Floor
Test 3	672	Loria Nursery	Ambiguous	11/21/2018 1:53:17 PM				5th Floor
Test 4	953	NICU 1	Ambiguous	11/26/2018 9:27:06 AM				5th Floor
Test 5	677	T5 Nursery	Boy	11/26/2018 10:57:42 AM				5th Floor

Active Transport

Name	ID	Room	Gender	Admitted	Suspend	Transport	Memo	Zone
Test 1	681	519	Boy	11/19/2018 11:28:50 AM				5th Floor
Test 2	632	Loria Nursery	Girl	11/19/2018 11:56:16 AM				5th Floor
Test 3	672	Loria Nursery	Ambiguous	11/21/2018 1:53:17 PM				5th Floor
Test 4	953	NICU 1	Ambiguous	11/26/2018 9:27:06 AM		1:14:49		5th Floor
Test 5	677	T5 Nursery	Boy	11/26/2018 10:57:42 AM				5th Floor

Transport Return

Name	ID	Room	Gender	Admitted	Suspend	Transport	Memo	Zone
Test 1	681	519	Boy	11/19/2018 11:28:50 AM				5th Floor
Test 2	632	Loria Nursery	Girl	11/19/2018 11:56:16 AM				5th Floor
Test 3	672	Loria Nursery	Ambiguous	11/21/2018 1:53:17 PM		1:4:41		5th Floor
Test 4	953	NICU 1	Ambiguous	11/26/2018 9:27:06 AM				5th Floor
Test 5	677	T5 Nursery	Boy	11/26/2018 10:57:42 AM				5th Floor

Transport Dialogue Box



Please note the
Duration field
format:

(HH:MM)

Transport - [tag name]

Specify the duration of the Transport then click OK to activate the Transport procedure for this tag.

Name [text box]

Duration (HH:MM) 00:05

Transport Zones


- All Zones
- Labor and Delivery
- Maternity

OK Cancel

How to change zones

Zones are a way of dividing the area covered by the Hugs system into sections and customizing the system alarm responses in each section to suit your security needs. The benefit is that users in one area only see alarms that occur in their section.

Zones cannot be used when common or shared doors are present.

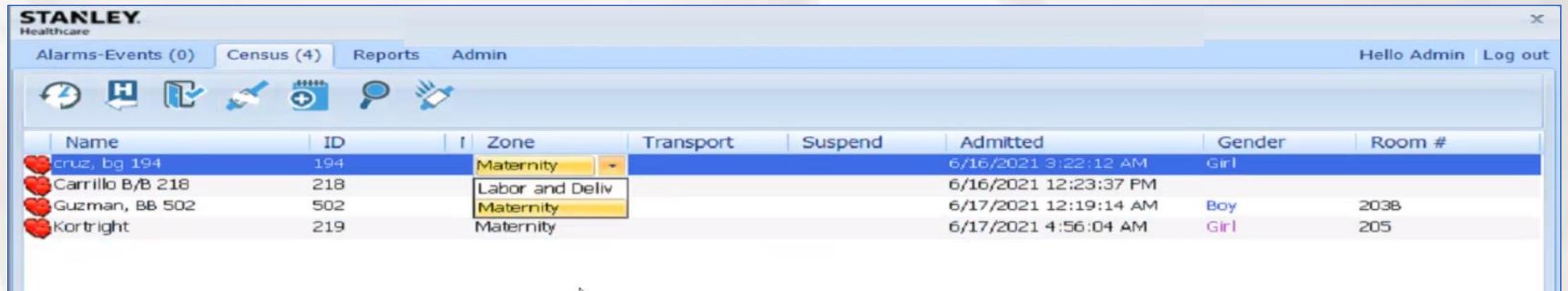

The screenshot shows the STANLEY Healthcare software interface. At the top, there are navigation tabs for "Alarms-Events (0)", "Census (4)", "Reports", and "Admin". The "Census (4)" tab is active. Below the tabs is a toolbar with various icons. The main area displays a table with the following columns: Name, ID, Zone, Transport, Suspend, Admitted, Gender, and Room #. The table contains four rows of patient data. The "Zone" column for the second row is highlighted, and a dropdown menu is open, showing "Labor and Deliv" as the selected option.

Name	ID	Zone	Transport	Suspend	Admitted	Gender	Room #
cruz, bg 194	194	Maternity			6/16/2021 3:22:12 AM	Girl	
Carrillo B/B 218	218	Labor and Deliv			6/16/2021 12:23:37 PM		
Guzman, BB 502	502	Maternity			6/17/2021 12:19:14 AM	Boy	203B
Kortright	219	Maternity			6/17/2021 4:56:04 AM	Girl	205

Zone information is updated automatically whenever a tag sees an Exciter, but you can also manually change a Zone location (if applicable.)

Alarms are reported in the zone to which the infant has been assigned.

Suspend Tamper

Name	ID	Room	Gender	Admitted	Suspend	Transport	Memo	Zone
Test 1	681	519	Boy	11/19/2018 11:28:50 AM				5th Floor
Test 2	632	Loria Nursery	Girl	11/19/2018 11:56:16 AM				5th Floor
Test 3	672	Loria Nursery	Ambiguous	11/21/2018 1:53:17 PM				5th Floor
Test 4	953	NICU 1	Ambiguous	11/26/2018 9:27:06 AM	1:42			5th Floor
Test 5	677	T5 Nursery	Boy	11/26/2018 10:57:42 AM				5th Floor

The Suspend Tamper commands Tamper Alarms for the selected tag for a period of 1 to 5 minutes (as set by the System Administrator) so that the band can be changed without generating a Tamper Alarm.

Name	ID	Room	Gender	Admitted	Suspend	Transport	Memo	Zone
Test 1	681	519	Boy	11/19/2018 11:28:50 AM				5th Floor
Test 2	632	Loria Nursery	Girl	11/19/2018 11:56:16 AM				5th Floor
Test 3	672	Loria Nursery	Ambiguous	11/21/2018 1:53:17 PM				5th Floor
Test 4	953	NICU 1	Ambiguous	11/26/2018 9:27:06 AM	1:42			5th Floor
Test 5	677	T5 Nursery	Boy	11/26/2018 10:57:42 AM				5th Floor

Note: *To avoid a Tamper alarm, the tag and the band must be re-applied before the Suspend time expires.*

Cleaning

STEP	ACTION
1	Remove all strap material.
2	Use a soft-bristle brush (e.g., a toothbrush) and a disinfectant soap or similar solution with no more than 20% alcohol .
3	Clean the Hugs Tag slots of strap debris. Pass the soft-bristle brush gently through the tag slot moving from the bottom to top of the Tag.
4	Dry the Tag thoroughly before reuse.

Do Not

- ❖ Use an autoclave to clean the Tags. Serious damage can result.
- ❖ Soak the Tags in alcohol or alcohol-based cleaners.
- ❖ Expose the Tags directly to X-rays or MRI scans.
- ❖ Immerse the Hugs Tag while cleaning.

Cleaning *continued*

Follow these rules for organizing Tags:

- When a Tag is removed and cleaned, return it for future use
- Use the supplied racks for tags
- Cycle the Tags: first in, first out
- Separate low battery Tags from Tags to be reused

Editing

Editing Name

The screenshot displays the STANLEY Healthcare software interface. It features a sidebar on the left with a list of test entries (Test 1 to Test 5). The main area shows a table of patient data. A context menu is open over the 'Test 3' row, with 'Edit Memo' highlighted. The interface includes navigation tabs (Alarms-Events, Census (5), Reports, Admin) and a user profile (Hello tpc | Log out).

Name	ID	Room	Gender	Admitted	Suspend	Transport	Memo	Zone
Test 1	681	519	Boy	11/19/2018 11:28:50 AM				5th Floor
Test 2	632	Loria Nursery	Girl	11/19/2018 11:56:16 AM				5th Floor
Test 3	672	Loria Nurser	Ambigious	11/19/2018 11:57:17 AM				5th Floor
Test 4	953	NICU 1	Ambigious	11/19/2018 11:57:17 AM				5th Floor
Test 5	677	T5 Nursery	Boy	11/19/2018 11:57:17 AM				5th Floor

Editing Room

Edit Memo

This is often used if you want to make a note regarding specific safety concerns or if you'll be transporting the baby and would like to share the location of the baby.

Important Safeguards

- Test all devices at least monthly
- Do not rely solely on the Hugs system to alert you
- Do not clear events or alarms without verifying the location and security of the infant
- Do not ignore the three security aspects to the Hugs system:
 - Exit
 - Tamper
 - Supervision Time Out

These alarms should receive priority intervention as they can indicate an abduction attempt.

- Do not ignore low battery alerts on your Hugs computer

Supervision of each tag by the system and by the facility's staff is the most important defense against portable electronic devices. The use of a security system does not replace the first line defense of a well-trained staff following proper internal security policies and procedures.

Contact your Hugs distributor before commencing with any construction projects or the installation of any wireless system in your building(s) to help reduce the risk of interference that could negatively impact your system's performance.

Time for practice.

Thank you for your time!

Please share your feedback with our manager, Lee Summerlin, at TPC Systems.

He can be reached via Lee.Summerlin@tpcsystems.com.